

# Surfax™

Disinfectant cleaner for floor  
and hard surfaces

**SURFAX™** is a colourless to pale yellow liquid with a characteristic mildly perfumed odour. It is an effective disinfectant cleaner based on a chlorine releasing action.



Cat. No.	Pack size
SFX0500	500 mL
SFX5000	5000 mL



Complies with the Biocidal Products Directive, 98/8/EC

## Composition

Minimum 0.6% w/v  
Sodium hypochlorite BP  
Stabilizers  
Perfume

## Activity

Broad spectrum :  
Bactericidal, Fungicidal & Virucidal

## Contact Time

10 minutes

## Features & Benefits

Rapid biocidal effect  
Bioburden tolerant  
Remove stains of blood and other body fluids  
Stabilized chlorine release formula  
Enhanced residual effect  
Non-corrosive-all surface compatible

## Applications

### MEDICAL

Disinfection of walls and floor of OT, maternity units, critical care areas, clinics, corridors, disposal areas, dressing rooms, toilets etc.

### INDUSTRY

Floor mopping & surface disinfection in pharmaceutical, food and dairy industry  
Dressing and laundry rooms, kitchen and laboratories  
Food processing areas

### GENERAL

General sanitation  
Floor & Toilets

## Usage Directions

### MEDICAL

HOSPITAL/LABORATORIES/TOILET/BATHROOMS/KITCHEN SURFACES  
– Apply undiluted **SURFAX™** onto the dirty area, leave for 10 minutes and wash.

FLOORS – Mop the floor with 20 mL of **SURFAX™** in half a bucket of water (4 Ltrs of water).

### INDUSTRY

Dilute 15 mL of **SURFAX™** in 1 Ltr of water for floor mopping & surface disinfection.